

Dräger DrugCheck™ Drug testing kit



ST-7055-2006

As part of diagnostic and forensic investigations, the Dräger DrugCheck™ enables the simultaneous qualitative detection of up to six classes of illegal substances in one sample of oral fluid. This very straightforward test is designed to be performed by anyone, anytime and anywhere.



ST-6968-2006

Dräger DrugCheck™:
Straightforward, reliable
and sensitive

Easy and reliable

Three simple steps – collect, test and store – are the trademark of the Dräger DrugCheck™. The testing kit comprises a sampler, test tube and test strip. Sampling is minimally invasive, does not embarrass the test subject unnecessarily, and permits precise control of the sampling process. This means that attempts at tampering can be more or less completely thwarted, establishing a basis of trust between user and test subject.

Hygiene first

The samplers are individually and hygienically packaged. When the oral fluid sampler is squeezed out into the test tube, it clips into place automatically, ensuring hygienic and drip-free disposal of the test kit components.

High sensitivity

Within just ten minutes, illegal substances (cocaine, opiates, amphetamines, methamphetamines, phencyclidine and cannabinoids) can be reliably detected using the test strip before they revert to their respective metabolites. Immunochemical detection of Δ^9 tetrahydrocannabinol (Δ^9 THC) – the psychoactive component in marijuana, hashish and hashish oil – reacts with a particularly high level of sensitivity.

Further advantages


As well as achieving very high sensitivity, the test offers the user many other advantages. The Dräger DrugCheck™ principle ensures that samples are analysed directly on site, though the oral fluid sample can also be preserved in the test tube, stored and sent to a laboratory at a later date for a confirmation analysis, if this is required.

ORDER INFORMATION


 Dräger DrugCheck™ (pack of 25)

83 19 550


Take sample

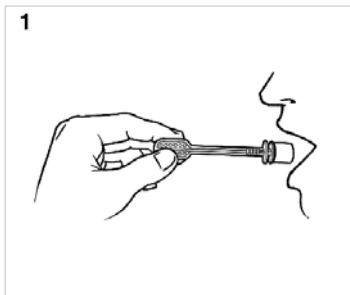


Insert sampler into tube extraction device

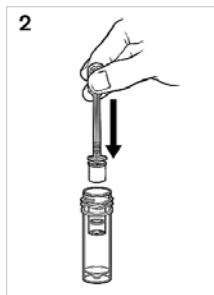


Squeeze out sampler

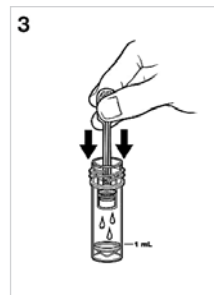


Remove sampler and extraction device

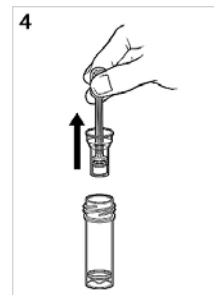


Remove test strip from packaging

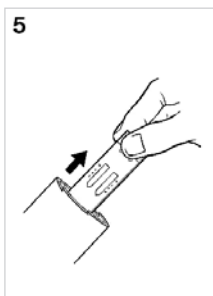

Insert test strip into tube

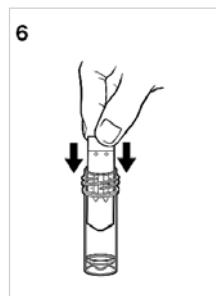


Read test strip
Examples A1 to A3

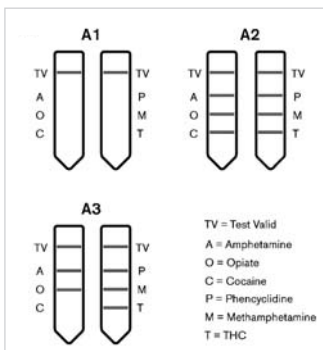

A1 = all drugs detected
A2 = no drugs detected
A3 = cocaine detected

Store sample for subsequent analysis if required

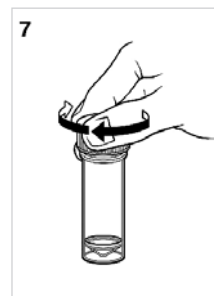

TECHNICAL DATA

Dräger DrugCheck™ limit value concentrations (cut-off values)

	Drug	ng/ml
COC	Cocaine	20
OPI	Opiate	40
AMP	Amphetamine	50
MET	Methamphetamine	50
PCP	Phencyclidine	10
THC	Δ9 Tetrahydrocannabinol	50

SUBSIDIARIES

AUSTRALIA

Dräger Safety Pacific Pty. Ltd.
Axxess Corporate Park
Mt. Waverley, Vic 3149
Tel +61 3 92 65 50 00
Fax +61 3 92 65 50 95

DENMARK

Dräger Safety Danmark A/S
Generatorvej 6 B, 2730 Herlev
Tel +45 4450 0000
Fax +45 4450 0001
www.draeger.com

NETHERLANDS

Dräger Safety Nederland B.V.
Edisonstraat 53
2700 AH Zoetermeer
Tel +31 79 344 46 66
Fax +31 79 344 47 90

SPAIN

Dräger Safety Hispania S.A.
Calle Xaudaró 5
28034 Madrid
Tel +34 91 728 34 00
Fax +34 91 729 48 99

BELGIUM

Dräger Safety Belgium S.A.
Heide 10
1780 Wommel, Belgium
Tel +32 24 62 62 11
Fax +32 24 62 62 01

FRANCE

Dräger Safety France S.A.S.
3c, Route de la Fédération
67025 Strasbourg Cedex
Tel +33 388 40 76 76
Fax +33 388 40 76 67

SINGAPORE

Dräger Safety Asia Pte. Ltd.
67 Ayer Rajah Crescent # 06 03
139950 Singapore
Tel +65 68 72 92 88
Fax +65 67 73 20 33

UNITED KINGDOM

Dräger Safety UK Ltd.
Blyth Riverside Business Park
Blyth, Northumberland NE24 4RG
Tel +44 1670 352 891
Fax +44 1670 356 266

P. R. CHINA

Beijing Fortune Draeger
Safety Equipment Co., Ltd.
Beijing 101300
Tel +86 10 80 49 80 00
Fax +86 10 80 49 80 05

ITALY

Dräger Safety Italiana SpA
Via Galvani 7
20094 Corsico (MI), Italy
Tel +39 02 48 69 71
Fax +39 02 48 60 24 64

REP. OF SOUTH AFRICA

Dräger South Africa (Pty) Ltd.
P.O.Box 68601
Bryanston 2021
Tel +27 11 465 99 59
Fax +27 11 465 69 53

Dräger Safety AG & Co. KGaA
Revalstrasse 1
23560 Luebeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
www.draeger.com